



Rabbit anti-CD14

Cat. No.: RBK033-05 (0.5 ml Concentrate)

Instructions for use

Intended use

This antibody is designed for the specific localisation of CD14 protein in formalin-fixed, paraffin-embedded tissue sections.

Anti-CD14 antibody is intended for in vitro diagnostic use.

Specifications

Specificity:	Human CD14
Clone:	EPR3653
Isotype:	Rabbit IgG
Species reactivity:	Human +, others not tested

Summary and Description

CD14 is a 55 kDa protein working as a receptor which binds the complex of lipopolysaccharids (LPS) and LPS-binding protein (LBP). It is mainly expressed on the surface of monocytes and macrophages. The staining of CD14 is especially helpful for the detection of normal and neoplastic cells of the monocytic cell lineage.

This anti-CD14 antibody stains monocytes/macrophages and Langerhans cells in Langerhans cell histiocytosis as well as tumour cells in monocytic leukaemia and histiocytic lymphomas. Additionally, the antibody reacts with diffuse large B-cell lymphomas and splenic marginal zone lymphoma but not with other B-cell lymphomas.

Reagent provided

Rabbit monoclonal antibody in PBS buffer pH 7.4 with carrier protein and preservative for stabilisation in the following formats:

Concentrate: 0.5 ml (Cat. No. RBK033-05)

Dilution of primary antibody

Dilution of Zytomed Systems' concentrated antibody depends on the detection system used. The final working dilution must always be determined by the user. The elaboration of staining protocol should be done by an experienced specialist. For Zytomed Systems' recommendations see chapter 'Staining procedure'.

Storage and handling

The antibody should be stored at 2-8°C without further dilution.

Dilutions of the concentrated antibody should be done in a suitable antibody dilution buffer (e.g. ZUC025 from Zytomed Systems). The diluted antibody should be stored at 2-8°C after use. The stability of this working solution depends on various parameters and has to be confirmed by appropriate controls.

The antibody provided is suitable for use until the expiry date indicated on the label, if stored at 2-8°C. Do not use product after the expiry date. Positive and negative controls should be run simultaneously with all specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact Zytomed Systems' technical support or your local distributor.

Precautions

Use through qualified personnel only.

Wear protective clothing to avoid contact of reagents and specimens with eye, skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with large amounts of water.

Microbial contamination of the reagent must be avoided, since otherwise non-specific staining may occur.

Sodium azide (NaN_3), used for stabilisation, is not considered hazardous material in the concentration used.

Reaction of sodium azide with lead or copper in drainage pipes can result in the formation of highly explosive metallic azides. Sodium azide should be discarded in a large volume of running water to avoid formation of deposits. A material safety data sheet (MSDS) for the pure substance is available upon request.

Staining procedure for formalin-fixed paraffin sections

Refer to the following table for conditions specifically recommended for this antibody. Also refer to detection system data sheets for guidance on specific staining protocols or other requirements.

Parameters	Zytomed Systems recommendations
*Pre-treatment	Heat Induced Epitope Retrieval (for example in Citrate Buffer pH 6.0 (ZUC028))
*Control tissue	Tonsil or diffuse large B-cell lymphoma
*Working dilution	1:50-1:200 (for concentrates)
*Incubation time	30 - 60 minutes

Quality control

The recommended positive control tissues for this antibody are tonsil or diffuse large B-cell lymphoma. We recommend carrying out a positive and a negative control with every staining run. Please refer to the instructions of the detection system for guidance on general quality control procedures.

Troubleshooting

If you observe unusual staining or other deviations from the expected results please read these instructions carefully, refer to the instructions of the detection system for relevant information or contact your local distributor.

Expected results

This antibody stains positive in the cytoplasmic membrane of monocytes and macrophages in formalin-fixed, paraffin-embedded tissue sections. Further details about the expression pattern of CD14 can be found in the chapter 'Summary and Description'. Interpretation of the staining results is solely the responsibility of the user. Any experimental result should be confirmed by a medically established diagnostic procedure.

Limitations of the Procedure

Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining, for example variations in fixation and embedding or the inherent nature of the tissue can cause inconsistent results (Nadji and Morales, 1983). Endogenous peroxidase, pseudoperoxidase activity in erythrocytes or biotin may cause non-specific staining depending on the detection system used. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive results with HRP (horse radish peroxidase) detection systems (Omata *et al*, 1980). Inadequate counterstaining and mounting can influence the interpretation of the results.

Zytomed Systems warrants that the product will meet all requirements described from its shipping date until the expiry date is reached, if the product is stored and utilised as recommended. No additional guarantees can be given. Under no circumstances shall Zytomed System be liable for any damages arising out of the use of the reagent provided.

Performance characteristics

Zytomed Systems has conducted studies to evaluate the performance of the antibody for use with a standard detection system. The product has been found to be sensitive and specific to the antigen of interest with minimal or no cross-reactivity.

Bibliography

Simiantonaki N *et al*. Int J Mol Med 20:21-29, 2007
Marmey B *et al*. Human Pathol 37:68-77, 2006
Raife TJ *et al*. Am J Clin Pathol 101:296-299, 1994
Nadji M, Morales AR. Ann N Y Acad Sci 420:134-138, 1983

Zhao D *et al*. Clin Cancer Res 13:2362-2368, 2007
Mason KD *et al*. Blood Rev 20:71-82, 2006
Wright SD *et al*. Science 249:1431-1433, 1990
Omata M *et al*. Am J Clin Pathol 73:626-632, 1980



www.zytomed-systems.de
Zytomed Systems GmbH •
Anhaltinerstraße 16 • 14163 Berlin,
Germany • Tel: (+49) 30-804 984 990

Explanations of the symbols on the product label

Symbols are used in accordance with ISO 15223-1.
Further symbols on the product label might be:



GSH07: Warning / Attention

RUO

For Research Use Only