

# CE IVD Rabbit anti-PAX-8 Cat. No.: RBK047-05 (0.5 ml Concentrate); RBG047 (6 ml Ready-to-use)

# Instructions for use

### Intended use

This antibody is designed for the specific localisation of PAX-8 protein in formalin-fixed, paraffin-embedded tissue sections.

Anti-PAX-8 antibody is intended for in vitro diagnostic use.

Specifications	
Specificity:	human PAX-8
Clone:	polyclonal
lsotype:	Rabbit Ig
Species reactivity:	Human $+$ , others not tested

#### Summary and Description

PAX-8 belongs to the protein family of paired box transcription factors. This nuclear protein is involved in thyroid follicular cell development and in expression of thyroid-specific genes. Mutations in PAX-8 gene have been associated with dysgenesis, follicular carcinoma and atypical adenoma of the thyroid gland.

The results of varies studies suggest that PAX-8 can also be a useful immunohistochemical marker for epithelial kidney and ovary carcinomas.

Carcinomas of breast and lung don't express PAX-8. Therefore PAX-8 in combination with organ-specific markers like mammaglobin, TTF-1 and Uroplakin-III can be helpful to determine the primary site of invasive micro-papillary carcinoma of the ovary from bladder, lung and breast.

# **Reagent provided**

Rabbit polyclonal antibody in PBS pH 7.4 including carrier proteins and preservative for stabilisation in the following formats:

Concentrate: 0.5 ml (Cat. No. RBK047-05) Ready-to-use: 6 ml (Cat. No. RBG047)

# Dilution of primary antibody

Dilution of Zytomed Systems' concentrated antibody depends on the detection system used. The final working dilution must always be determined by the user. The validation of staining protocol should be done by an experienced specialist. For Zytomed Systems' recommendations see chapter 'Staining procedure'.

#### Storage and handling

The antibody should be stored at 2-8°C without further dilution.

Dilutions of the concentrated antibody should be done in a suitable antibody dilution buffer (e.g. ZUC025 from Zytomed Systems). The diluted antibody should be stored at 2-8°C after use. The stability of this working solution depends on various parameters and has to be confirmed by appropriate controls.

The antibody provided is suitable for use until the expiry date indicated on the label, if stored at 2-8°C. Do not use product after the expiry date. Positive and negative controls should be run simultaneously with all specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact Zytomed Systems' technical support or your local distributor.

# Precautions

Use through qualified personnel only.

Wear protective clothing to avoid contact of reagents and specimens with eye, skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with large amounts of water.

Microbial contamination of the reagent must be avoided, since otherwise non-specific staining may occur. Sodium azide (NaN<sub>3</sub>), used for stabilisation, is not considered hazardous material in the concentration used. Reaction of sodium azide with lead or copper in drainage pipes can result in the formation of highly explosive metallic azides. Sodium azide should be discarded in a large volume of running water to avoid formation of deposits. A material safety data sheet (MSDS) for the pure substance is available upon request.

#### Staining procedure

Refer to the following table for conditions specifically recommended for this antibody. Also refer to detection system data sheets for guidance on specific staining protocols or other requirements.

Parameters \*Pre-treatment \*Control tissue \*Working dilution \*Incubation time

Zytomed Systems recommendations Heat Induced Epitope Retrieval (for example in Citrate Buffer pH 6.0) Ovary carcinoma (non-mucinous) or thyroid carcinoma 1:25 - 1:50 (for concentrates) 60 minutes

#### **Quality control**

The recommended positive control tissues for this antibody are non-mucinous ovary carcinoma or thyroid carcinoma. We recommend carrying out a positive and a negative control with every staining run. Please refer to the instructions of the detection system for guidance on general guality control procedures.

#### Troubleshooting

If you observe unusual staining or other deviations from the expected results please read these instructions carefully, refer to the instructions of the detection system for relevant information or contact your local distributor.

#### **Expected results**

This antibody stains positive in the nuclei of PAX-8 positive cells in formalin-fixed, paraffin-embedded tissue sections. Further details about the expression pattern of PAX-8 can be found in the chapter 'Summary and Description'. Interpretation of the staining results is solely the responsibility of the user. Any experimental result should be confirmed by a medically established diagnostic procedure.

#### Limitations of the Procedure

Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining, for example variations in fixation and embedding or the inherent nature of the tissue can cause inconsistent results (Nadji and Morales, 1983). Endogenous peroxidase, pseudoperoxidase activity in erythrocytesor biotin may cause non-specific staining depending on the detection system used. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive results with HRP (horse radish peroxidase) detection systems (Omata et al, 1980). Inadequate counterstaining and mounting can influence the interpretation of the results.

Zytomed Systems warrants that the product will meet all requirements described from its shipping date until the expiry date is reached, if the product is stored and utilised as recommended. No additional guarantees can be given. Under no circumstances shall Zytomed System be liable for any damages arising out of the use of the reagent provided.

#### **Performance characteristics**

Zytomed Systems has conducted studies to evaluate the performance of the antibody for use with a standard detection system. The product has been found to be sensitive and specific to the antigen of interest with minimal or no cross-reactivity.

#### Bibliography

Lotan TL et al. Am J Surg Pathol 33:1037-1041, 2009 Nonaka D et al. Mod Pathol 21:192-200, 2008 Bowen NJ et al. Gynecol oncol 104:331-337, 2007 Tina DP et al. J Biol Chem 278:3395-3402, 2003 Nikiforova MN et al. Am J Surg Pathol 26:1016-1023, 2002 Omata M et al. Am J Clin Pathol 73:626-632, 1980

Tong GX et al. Mod Pathol 22:1218-1227, 2009 Nonaka D et al. Am J Surg Pathol 32:1566-1571, 2008 Zhang P et al. Pathol Int 56:240-245, 2006 Marques AR et al. J Clin Endocrinol Metab 87:3947-3952, 2002 Nadji M, Morales AR. Ann N Y Acad Sci 420:134-138, 1983



www.zytomed-systems.de

Zytomed Systems GmbH • Anhaltinerstraße 16 • 14163 Berlin, Germany • Tel: (+49) 30-804 984 990

#### Explanations of the symbols on the product label

Symbols are used in accordance with ISO 15223-1. Further symbols on the product label might be:



GSH07: Warning / Attention

RŮO

For Research Use Only