



## Rabbit anti-PD-L1 (CD274)

Cat. No.: RBK063-05 (0.5 ml Concentrate);  
RBG063 (6 ml Ready-to-use)

### Instructions for use

#### Intended use

This antibody is designed for the specific localisation of PD-L1 (CD274), in formalin-fixed, paraffin-embedded tissue sections.

Anti-PD-L1 antibody is intended for in vitro diagnostic use.

#### Specifications

<b>Specificity:</b>	Human PD-L1 protein
<b>Clone:</b>	CAL10
<b>Isotype:</b>	Rabbit IgG
<b>Species reactivity:</b>	Human +, others not tested.

#### Summary and Description

PD-L1 (Programmed Cell Death Ligand 1, also known as CD274) is a receptor ligand which is expressed by hematopoietic and non-hematopoietic cells, such as T- and B-lymphocytes and various types of tumor cells. PD-L1 is a type-I transmembrane protein.

Binding of PD-L1 to its receptor PD1 inhibits T-cell activation and cytokine production. This mechanism renders PD-L1 overexpressing tumor cells resistant to T cell-mediated lysis. Immunohistochemical detection of the PD-L1 protein is of great importance for the use of novel anticancer drugs such as checkpoint inhibitors.

#### Reagent provided

Rabbit monoclonal antibody in buffer with carrier protein and preservative for stabilisation in the following formats:

<b>Concentrate:</b>	0.5 ml	(Cat. No. RBK063-05)
<b>Ready-to-use:</b>	6 ml	(Cat. No. RBG063)

#### Dilution of primary antibody

Dilution of Zytomed Systems' concentrated antibody depends on the detection system used. The final working dilution must always be determined by the user. The validation of staining protocol should be done by an experienced specialist. For Zytomed Systems' recommendations see chapter 'Staining procedure'.

#### Storage and handling

The antibody should be stored at 2-8°C without further dilution.

Dilutions of the concentrated antibody should be done in a suitable antibody dilution buffer (e.g. ZUC025 from Zytomed Systems). The diluted antibody should be stored at 2-8°C after use. The stability of this working solution depends on various parameters and has to be confirmed by appropriate controls.

The antibody provided is suitable for use until the expiry date indicated on the label, if stored at 2-8°C. Do not use product after the expiry date. Positive and negative controls should be run simultaneously with all specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact Zytomed Systems' technical support or your local distributor.

#### Precautions

Use through qualified personnel only.

Wear protective clothing to avoid contact of reagents and specimens with eye, skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with large amounts of water.

Microbial contamination of the reagent must be avoided, since otherwise non-specific staining may occur.

Sodium azide ( $\text{NaN}_3$ ), used for stabilisation, is not considered hazardous material in the concentration used.

Reaction of sodium azide with lead or copper in drainage pipes can result in the formation of highly explosive metallic azides. Sodium azide should be discarded in a large volume of running water to avoid formation of deposits. A material safety data sheet (MSDS) for the pure substance is available upon request.

## Staining procedure

Refer to the following table for conditions specifically recommended for this antibody. Also refer to detection system data sheets for guidance on specific staining protocols or other requirements.

Parameters	Zytomed Systems recommendations
*Pre-treatment	Heat Induced Epitope Retrieval (for example in Citrate Buffer pH 6.0)
*Control tissue	Adenocarcinoma of the lung or tonsil
*Working dilution	1:100-1:200 (for concentrates)
*Incubation time	60 minutes

## Quality control

The recommended positive control tissues for this antibody are adenocarcinoma of the lung or tonsil. We recommend carrying out a positive and a negative control with every staining run. Please refer to the instructions of the detection system for guidance on general quality control procedures.

## Troubleshooting

If you observe unusual staining or other deviations from the expected results please read these instructions carefully, refer to the instructions of the detection system for relevant information or contact your local distributor.

## Expected results

This antibody stains positive in the cytoplasmic membrane and in the cytoplasm of PD-L1 positive cells in formalin-fixed, paraffin-embedded tissue sections. Further details about the expression pattern of PD-L1 (CD274) can be found in the chapter 'Summary and Description'. A weak staining of endothelial cells can be observed occasionally. Interpretation of the staining results is solely the responsibility of the user. Any experimental result should be confirmed by a medically established diagnostic procedure.

## Limitations of the Procedure

Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining, for example variations in fixation and embedding or the inherent nature of the tissue can cause inconsistent results (Nadji and Morales, 1983). Endogenous peroxidase, pseudoperoxidase activity in erythrocytes or biotin may cause non-specific staining depending on the detection system used. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive results with HRP (horse radish peroxidase) detection systems (Omata *et al*, 1980). Inadequate counterstaining and mounting can influence the interpretation of the results.

Zytomed Systems warrants that the product will meet all requirements described from its shipping date until the expiry date is reached, if the product is stored and utilised as recommended. No additional guarantees can be given. Under no circumstances shall Zytomed System be liable for any damages arising out of the use of the reagent provided.

## Performance characteristics

Zytomed Systems has conducted studies to evaluate the performance of the antibody for use with a standard detection system. The product has been found to be sensitive and specific to the antigen of interest with minimal or no cross-reactivity.

## Bibliography

Park IH *et al*. Clin Breast Cancer 16:51-58, 2016  
Singh BP, Salama AK. Cancers (Basel) 8(1):17, 2016  
Patel SP, Kurzrock R. Mol Cancer Ther. 14:847-856, 2015  
Ostrand-Rosenberg S *et al*. J Immunol 193:3835-3841, 2014  
Omata M *et al*. Am J Clin Pathol 73:626-632, 1980

Xia B, Herbst RS. Immunotherapy 8:279-298, 2016  
Tokito T *et al*. Eur J Cancer 55:7-14, 2016  
Kakavand H *et al*. Mod Pathol 28:1535-1544, 2015  
Nadji M and Morales AR. Ann NY Acad Sci 420:134-139, 1983



[www.zytomed-systems.de](http://www.zytomed-systems.de)

Zytomed Systems GmbH • Anhaltinerstraße 16 •  
14163 Berlin, Germany • Tel: (+49) 30-804 984 990

## Explanations of the symbols on the product label

Symbols are used in accordance with ISO 15223-1. Further symbols on the product label might be:



GSH07: Warning / Attention

For Research Use Only