

ZytoMed Systems Reagents on Roche Instruments

22.09.2020



BM Ultra = BenchMark Ultra®

BM XT = BenchMark XT®

BenchMark® and NexES® as well as iView™, OptiView™ and UltraView™ are registered trademarks of Roche Medical Systems, Inc.

Description	Cat. No.	Dilution	Enzyme	Pre-fix	CC1	CC2	Incubation Ab	Enhancer	UltraWash	Detection	Instrument
Actin alpha (Smooth Muscl	MSK030	1:100			8 min		16 min 37°C			UltraView	BM
Actin alpha (Smooth Muscl	MSK030	1:100			36 min		32 min 37°C			iView DAB	BM XT
Actin alpha (Smooth Muscl	MSK030	1:200			60 min		32 min 37°C			Ultra View DAB	BM XT
Actin alpha (Smooth Muscl	MSK030	1:200			60 min		32 min 37°C			iView DAB	BM XT
Actin alpha (Smooth Muscl	MSK030	1:300			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
Actin alpha (Smooth Muscl	BMS001	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
Bcl-2	RBK016	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
BRAF V600E	ab228461	1:50			30 min		16 min 37°C	?	?	ChromoMap DAB	Discovery XT
BRAF V600E	ab228461	1:400			52 min		60 min 37°C	yes		Ultra View Red AP	BM Ultra
Calretinin	RBK003	1:200			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
Calretinin	RBK003	1:200			36 min		32 min 37°C			iView DAB	
Calretinin	BRB022	RTU			60 min		28 min 37°C			Ultra View DAB	BM XT
CD3	RBK024	1:400			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
CD4	BRB042	RTU			60 min		32 min 37°C		yes	Ultra View DAB	BM XT
CD5	RBK023-05	1:50			30 min		32 min 37°C			Ultra View DAB	BM XT
CD5	RBK023-05	1:50			30 min		32 min 37°C			iView DAB	BM XT
CD8	BRB036	RTU			36 min		8 min 37°C			Ultra View DAB	BM Ultra
CD8	MSK011	1:50			8 min		16 min 37°C			UltraView	BM
CD10	MSK070	1:25			8 min		16 min 37°C			UltraView	BM
CD10	MSK070	1:200			60 min		32 min 37°C	yes		UltraView APRed	
CD10	BMS043	RTU			90 min		28 min 37°C			Ultra View DAB	BM XT
CD20	BMS003	RTU			60 min		10 min 37°C		yes	Ultra View Red AP	
CD20	MSK008	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
CD21	RBK036	1:50			8 min		16 min 37°C			UltraView	BM
CD31	BMS044	RTU			60 min		12 min 37°C		yes	Ultra View DAB	BM XT
CD34	BMS045	RTU			30 min		20 min 37°C		yes	Ultra View DAB	BM XT
CD45 LCA	MSK054	1:400			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
CD45 LCA	MSK054	1:50			8 min		16 min 37°C			UltraView	BM
CD45 LCA	MSK054	1:50			30 min		32 min 37°C			iView	BM

CD45 LCA	BMS046	RTU			30 min		8 min 37°C			Ultra View DAB	BM XT
CD56	RBK050	1:400			36 min		32 min 37°C				BM XT
CD56	RBK050	1:500			36 min		32 min 37°C				BM XT
CD56	BRB039	RTU			36 min		20 min 37°C			Ultra View DAB	BM Ultra
CD68	MSK055	1:1000			60 min		30 min 37°C			Ultra View	BM XT
CD68	MSK055	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
CD79a	BMS005	RTU			60 min		32 min 37°C		yes	Ultra View Red AP	
CD79a	MSK036	1:50			8 min		16 min 37°C			UltraView	BM
CD138	MSK063	1:200			8 min		16 min 37°C			UltraView	BM
CD138	MSK063	1:200			64 min		32 min 37°C			Ultra View Red AP	BM XT
CDX2	RBK019	1:50				8 min	32 min 37°C	yes		Ultra View DAB	
CDX2	BRB028	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
CDX2	RBK019	1:50			30 min		32 min 37°C			UltraView	BM
CDX2	RBK019	1:50			30 min		32 min 37°C			iView	BM
CDX2	RBK019	1:100			30 min		60 min 37°C			Ultra View DAB	BM XT
CDX2	RBK019	1:100			60 min		32 min 37°C			Ultra View DAB	BM XT
CDX2	RBK019	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
CEA	BMS029	RTU			20 min		20 min 37°C			Ultra View DAB	BM Ultra
CEA	MSG022	RTU			60 min		6 min 37°C		yes	Ultra View Red AP	
CEA	MSK022	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
Chromogranin A	BMS018	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
Chromogranin A	MSK062	1:50			36 min		28 min 37°C		yes	Ultra View DAB	BM Ultra
Chromogranin A	MSK062	1:200			30 min		32 min 37°C	yes		Ultra View DAB	BM XT
Chromogranin A	MSK062	1:300			30 min		32 min 37°C			iView DAB	BM XT
CK (AE1/AE3)	MSK019	1:80			36 min		32 min 37°C			iView DAB	BM XT
CK (AE1/AE3)	MSK019	1:100			36 min		32 min 37°C			iView DAB	BM XT
CK (AE1/AE3)	MSK019	1:200			90 min		20 min 37°C			Ultra View DAB	
Cytokeratin (HMW)	BMS015	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
Cytokeratin (HMW)	MSK027	1:100	8 min Protease 1				32 min 37°C		yes	Ultra View DAB	BM Ultra
CK pan plus	BMS057	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
CK pan plus	MSK098-05	1:400			36 min		40 min 37°C			Ultra View DAB	BM Ultra
CK pan plus	MSK098-05	1:400			32 min		40 min 37°C			OptiView DAB	BM Ultra
CMV	MSG121	RTU	30 min				32 min 37°C			ja	Ultra View Red AP
Cytokeratin 5 & 6	BMS017	RTU			64 min		32 min 37°C			Ultra View DAB	BM Ultra
Cytokeratin 5 & 6	MSK034	1:100					32 min 37°C				Nexus
Cytokeratin 5 & 6	MSK034	1:50			30 min		32 min 37°C			UltraView	BM
Cytokeratin 5 & 14	MSK106-05	1:100			36 min		32 min 37°C			Ultra View DAB	BM Ultra
Cytokeratin 7	BMS030	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
Cytokeratin 7	MSK032	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra

Cytokeratin 7	MSK032	1:200			36 min		32 min 37°C			iView DAB	BM XT
Cytokeratin 18	MSK016	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
Cytokeratin 19	MSK017	1:200			8 min		16 min 37°C			UltraView	BM
Cytokeratin19	MSK017	1:200			30 min		32 min 37°C			iView	BM
Cytokeratin19	MSK017	1:300			30 min		32 min 37°C			iView DAB	BM XT
Cytokeratin 20	BMS037	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
Desmin	BMS007	RTU			60 min		28 min 37°C		yes	Ultra View Red AP	
Desmin	MSK053	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
Desmin	MSK053	1:400			60 min		32 min 37°C			iView DAB	BM XT
E-Cadherin	BRB047	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
E-Cadherin	MSK033	1:10			30 min		20 min 37°C			Ultra View DAB	
E-Cadherin	MSK033	1:80			60 min		28 min 37°C			Ultra View DAB	BM XT
E-Cadherin	MSK033	1:80			60 min		32 min 37°C			iView DAB	BM XT
E-Cadherin	MSK033	1:100			36 min		28 min 37°C		yes	Ultra View DAB	BM Ultra
EGFr	MSK014	1:100	P1 8 min				30 min 37°C			Ultra View	BM XT
EMA	MSK059	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
EMA	MSK059	1:200			8 min		16 min 37°C			UltraView	BM
EMA	MSK059	1:200			30 min		32 min 37°C			iView	BM
EMA	MSK059	1:400			?	?	20 min 37°C			iView DAB	BM XT
Ep-CAM, ESA (Ber-EP4)	505-11171	RTU			30 min		32 min 37°C	yes		Ultra View DAB	BM XT
Ep-CAM, ESA (Ber-EP4)	BMS048	RTU	P1 8 min				28 min 37°C		yes	Ultra View DAB	BM XT
ER (SP1)	RBK018	1:50			30 min		57 min 37°C			Ultra View DAB	
ER (SP1)	RBK018	1:200			60 min		32 min 37°C			Ultra View DAB	BM XT
ER (SP1)	RBK018	1:200			52 min		28 min 37°C		yes	Ultra View DAB	BM Ultra
ER (SP1)	BRB053	RTU			60 min		28 min 37°C		yes	Ultra View DAB	BM XT
ERG	MSK090	?			52 min		32 min 37°C			iView	
ERG	MSK090	?			30 min		16 min 37°C			Ultra View	
GATA3	MSK100-05	1:100			30 min		24 min 37°C			Ultra View DAB	BM Ultra
GATA3	BMS054	RTU			64 min		32 min 37°C			Ultra View DAB	BM Ultra
Glypican-3	MSG067	RTU			60 min		30 min 37°C		yes	Ultra View Red AP	
Glypican-3	MSK067	1:100			8 min		16 min 37°C			UltraView	BM
Glypican-3	MSK067	1:100			30 min		40 min 20°C		yes	Ultra View DAB	BM XT
Glypican-3	MSK067	1:100			60 min		32 min 37°C			Ultra View DAB	BM XT
Glypican-3	MSK067	1:100			60 min		32 min 37°C			iView DAB	BM XT
Glypican-3	MSK067	1:200			60 min		32 min 37°C			Ultra View DAB	BM XT
Glypican-3	MSK067	1:200			60 min		32 min 37°C			iView DAB	BM XT
Glypican-3	MSK067	1:200			90 min		32 min 37°C			Ultra View DAB	BM XT
Helicobacter	RBK012	1:600			8 min		32 min 37°C		yes	Ultra View Red AP	
Helicobacter	RBK012	1:200			30 min		32 min 37°C			UltraView	BM
Helicobacter	RBK012	1:200			30 min		32 min 37°C			iView	BM

Helicobacter	RBK012	1:200			8 min		32 min 37°C			iView DAB	BM XT
Helicobacter	RBK012	1:200			60 min		32 min 37°C			iView DAB	BM XT
Her2 (SP3)	RBK026	1:75				30 min	32 min 37°C			Ultra View DAB	BM XT
Her2 (SP3)	RBK026	1:100			36 min		32 min 37°C		yes	Ultra View DAB	BM Ultra
Her2 (SP3)	RBK026	1:50			60 min		60 min			Ultra View Red AP	BM Ultra
Her2 (SP3)	RBK026	1:100			36 min		20 min 37°C		yes	Ultra View DAB	BM Ultra
Her2 (SP3) stomach	RBK026	1:25			30 min		32 min 42°C		yes	iView DAB	BM Ultra
IgG4	MSK084	1:200				30 min	30 min 37°C			Ultra View DAB	BM XT
IgG4	MSK084	1:50	P1 / 6 min				30 min 37°C			Ultra View DAB	BM
Ki-67	BMS009	RTU			60 min		32 min 37°C		yes	Ultra View Red AP	
Ki-67	MSK018	1:200			60 min		40 min 37°C			iView DAB	BM XT
Ki-67	MSK018	1:250			32 min		20 min 37°C			OptiView DAB	BM Ultra
Ki-67	MSK018	1:300			60 min		32 min 37°C			iView DAB	BM XT
Ki-67	MSK018	1:400			60 min		32 min 37°C			Ultra View DAB	BM XT
Ki-67	MSK018	1:400			36 min		32 min 37°C			iView DAB	BM Ultra
Ki-67	MSK018	1:400			36 min		32 min 37°C		yes	Ultra View DAB	BM Ultra
Ki-67	MSK018	1:600			60 min		32 min 37°C		yes	Ultra View Red AP	
Ki-67 (SP6)	RBK027	1:200			36 min		24 min 37°C		yes	Ultra View DAB	BM
Ki-67 (SP6)	BRB040	RTU			60 min		28 min 37°C		yes	Ultra View DAB	BM XT
Ksp-Cadherin	MSK068	1:100			8 min		16 min 37°C			UltraView	BM
LYVE1	RBK014	1:50			60 min		44 min 42°C	yes		Ultra View DAB	
Mammaglobin	RBK022-05	1:100			8 min		16 min 37°C			UltraView	BM
Mammaglobin	RBK022-05	1:100			30 min		32 min 37°C			iView	BM
MDM2	113-0230	1:25			36 min		60 min 37°C	nein	nein	Ultra View DAB	BM XT
Melan-A / MART-1	MSK056	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
Melanoma / HMB45	BMS010	RTU			60 min		24 min 37°C		yes	Ultra View Red AP	
Melanoma / HMB45	MSK007	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
MLH1	MSK045	1:25		x*		90 min	180 min 37°C	yes	no!	Ultra View DAB v3	BM XT
MLH1	MSK045	1:25		x	90 min		60 min 37°C	yes	no!	Ultra View DAB	BM
MSH2	MSG031	RTU			30 min		32 min 37°C	yes	yes	Ultraview Red	BM XT
Myosin, SM	MSK081-05	1:100			8 min		16 min 37°C			UltraView	BM
Myosin, SM	MSK081-05	1:100			30 min		32 min 37°C			iView	BM
Oct-2	MSK073-05	1:50			30 min		32 min 37°C			iView	BM
p16 ^{INK4a}	MSK123-05	1:100			64 min		60 min 37°C		yes	Ultra View DAB	BM
p16 ^{INK4a}	MSK123-05	1:200			64 min		32 min 37°C			Ultra View DAB	BM
p40 (BC28)	BMS050	RTU			60 min		28 min 37°C			Ultra View DAB	BM XT
p40 (polyklonal)	RBK054-05	1:100			32 min		26 min 37°C			Ultra View DAB v3	BM XT
P504S	BRB035	RTU			30 min		20 min 37°C			Ultra View DAB	BM XT
P504S + p40	K002-05 + RBK054	both 1:100			60 min	double stain	44 min 37°C		yes	Ultra View Red3	
p53	MSK020	1:100			36 min		28 min 37°C		yes	Ultra View DAB	BM Ultra

p53	MSK020	1:200			36 min		32 min 37°C			iView DAB	BM XT
p63	BMS052	RTU			36 min		32 min 37°C			Ultra View DAB	BM Ultra
p63	MSK087	1:200			60 min		32 min 37°C			Ultra View Red AP	
Pan TRK	RBK069-05	1:80			64 min		60 min 37°C	yes		OptiView	BM Ultra
Pax-5	RBK008-05	1:20			60 min		32 min 37°C			OptiView	
Pax-8	RBK047-05	1:50			60 min		60 min 37°C			Ultra View DAB	BM XT
PD-L1	RBK063	1:200			32 min		32 min 37°C			OptiView DAB	BM Ultra
PIN Cocktail	CO001K	1:50			52 min		32 min 37°C		yes	Ultra View DAB	BM Ultra
PIN Cocktail	CO001K	1:50			60 min		60 min 37°C			Ultra View AP Red	BM XT
PIN Cocktail	CO001K	1:50			60 min		32 min 37°C			iView DAB	BM XT
PIN Cocktail	CO001K	1:75			90 min		32 min 37°C			iView DAB	BM XT
PIN Cocktail	CO001K	1:80			60 min		32 min 37°C			iView DAB	BM XT
PMS2	MSK064-05	1:25				60 min	32 min 37°C	yes		iView DAB	BM XT
PMS2	MSK064-05	1:50			8 min		44 min RT				
PMS2	RBG048	RTU			30 min		32 min 37°C	yes		Ultra View DAB	BM XT
PMS2	MSK064-05	1:50				36 min	44 min RT			Ultra View DAB	
Podoplanin / D2-40	MSK057	1:50			30 min		32 min 37°C			UltraView	BM
Podoplanin / D2-40	MSK057	1:50			30 min		32 min 37°C			iView	BM
Podoplanin / D2-40	MSK057	1:50					32 min 37°C				Nexus
PR (SP42)	RBK020	1:100			30 min		57 min 37°C			Ultra View DAB	
PR (SP42)	RBK020	1:100			52 min		28 min 37°C		yes	Ultra View DAB	BM Ultra
PR (SP42)	RBK020	1:200			30 min		32 min 37°C			Ultra View DAB	BM XT
PR (SP42)	RBK020	1:200			52 min		32 min 37°C	yes		Ultra View DAB	BM Ultra
PR (SP42)	RBK020	1:200			60 min		32 min 37°C			Ultra View DAB	BM XT
PR (SP42)	RBK020	1:200			30 min		32 min 37°C			iView DAB	BM XT
PSA	MSK051	1:200			8 min		16 min 37°C			UltraView	BM
PSA	MSK051	1:200			30 min		32 min 37°C			iView	BM
PSA	MSK051	1:400			36 min		32 min 37°C		yes	Ultra View DAB	BM Ultra
S100	BMS013	RTU			60 min		8 min 37°C		yes	Ultra View Red AP	
S100	MSK050	1:100			36 min		24 min 37°C		yes	Ultra View DAB	BM Ultra
SOX10	MSK117	1:100			60 min		28 min 37°C			Ultra View DAB	BM Ultra
SOX10	MSG117	RTU			32 min		16 min 37°C			OptiView DAB	BM Ultra
SOX10	MSG117	RTU			64 min		32 min 37°C			Ultraview DAB	BM XT
SOX10	MSG117	RTU			64 min		32 min 37°C			Ultraview DAB	BM Ultra
SSTR2A	RBK046-05	1:100			32 min		24 min 37°C		no!	Ultra View AP Red	BM Ultra
Synaptophysin	RBK011	1:100			36 min		28 min 37°C		yes	Ultra View DAB	BM Ultra
Synaptophysin	519-2174	1:600			60 min		32 min 37°C		yes	Ultra View Red AP	
Synaptophysin	RBK011	1:400			30 min		32 min RT			Ultra View DAB	BM Ultra

